

PHITEN SUPPORTER SHOULDER_MIDDLE TYPE

INSTRUCTION

Thank you for making the purchase of this product.
Please read the instruction carefully before usage.
Also, please keep the instruction in a safe place in order to refer anytime.

USE Protecting the shoulder.

FEATURE

A shoulder supporter adjustable for wide range of sizes and fixing strength.

1.The resin stay applied in the front and back supports to raise the arm, and also prevents the back and forth movements at the same time. 2.This product realized the both of well-fit and support strength by Phiten's unique designing. 3.The upper arm area of this supporter is available to be adjusted by the Velcro fastener.


SIZE

	M	L
Chest	75~95cm	90~120cm


- * Measure the chest perimeter under the armpit.
- * This product is applicable to both of right and left shoulders.
- * Use the bigger one if your chest perimeter fits two sizes.

HOW TO

* The following instruction is for the right shoulder.

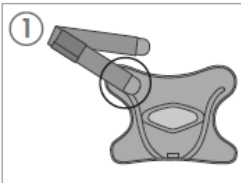

Apply the belt on the upper-left surface of the main body.

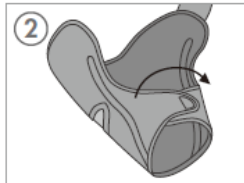

Reverse the product and fix the Velcro fastener fitting the size of upper arm.




Put the arm through the supporter, and adjust the center part coming on the tip of shoulder.



Fix the Velcro fastener in front of the chest and adjust the armpit pad. Do not twist the belt.




Adjust the fixing-strength of the Velcro fastener and re-position armpit pad.

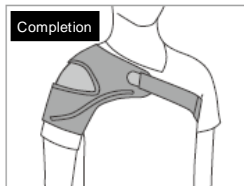


Completion

SURFACE

TOP

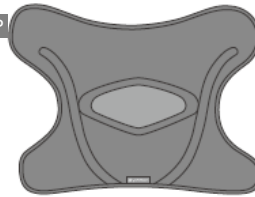

BOTTOM

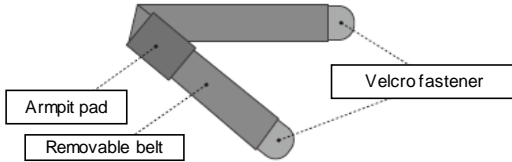

Armpit pad

Removable belt

Velcro fastener

BACK

TOP

The resin stay could be removed from this hole.

BOTTOM

Velcro fastener of the upper arm

Removable resin stay

About the removable resin stay

- It is recommended to use the resin stay according to the shoulder condition.
- *To fix the shoulder more heavily: Use both stays.
 - *To fix the shoulder moderately: Use the back stay only.
 - *To allow the shoulder movement: Do not use both stays.

CAUTION ON

*This product is only aimed for protecting the area of concern and not for medical treatment purpose. *Refrain from usage if there is injury, the fracture, wound, lump, eczema, or rash on the applied part. *Stop using immediately if allergies or any abnormality such as eczema appears, and consult with physician. *Use the product only on the intended area. *Do not use the product when sleeping. *When using the product for long time, loosen and re-tighten the product time by time *Do not use this product turned the inside out or upside down. *Avoid over-tightening the area with product. *Be sure to relocate the product to the proper position if it gets loose or moving. *If the abrasion or damage is found, or if Velcro fastener no longer works, stop using immediately. *Prevent the Velcro fastener from touching the fabric as it may cause fray or ladder.

<Care and Cleaning >

- * Do not tumble dry. * Do not bleach.
- * Wash this product separately from white or light color clothes to avoid color migration.
- * Do not leave this product being wet or in water for long time.
- * Tie the Velcro fastener to the main body before washing, to avoid deterioration of the function of the Velcro fastener.



Main body (Outside surface), Armpit pad (Inner surface): Nylon93%, Polyurethane7% / Main body (Inner surface), shoulder part of the main body, armpit pad (Outside surface): Nylon80%, Polyurethane 20% / Removable belt: Nylon90%, Polyurethane10% / Velcro fastener: Polyester 100% / Resin stay: Polyethylene



AQUA TITAN is the brand of water-soluble metal technology developed by Phiten.



AQUA PALLADIUM is the brand of water-soluble metal technology developed by Phiten.

MADE IN JAPAN

*Please note that the specifications are subject to change without notice.

Manufactured and distributed by Phiten Co., Ltd.
490-3 Hatukashi Hishikawa-cho, Fushimi-ku, Kyoto-city
Customer hotline: 0120-510702 / AM10:00 to PM5:00 (except Sat, Sun and public holidays)
URL <http://www.phiten.com/>